

# QuadraLite™ Anaesthetic Face Mask User Guide

1




Select the correct QuadraLite size for the patient using the size selection guide. Four sizes provide a solution for all patients.

Patient	Age	Weight (Kg)	Traditional mask size	QuadraLite™ masks	
Neonatal	Preterm	< 2	0	 Code: 7191000	
	Term	2 – 4			
Infant	3 months	6	1		
	6 months	8			
	9 months	9			
Paediatric	1 year	10	2		 Code: 7192000
	2 years	12			
	3 years	14			
	4 years	16			
	5 years	18			
Small Adult	6 years	20	3		
	7 years	23			
	8 years	26			
	9 years	28			
	10 years	32			
Medium Adult	11 years	35	4	 Code: 7193000	
	12 years	39			
	13 years	44			
	14 years	50			
Large Adult	15 years	54	5		
	16 years	58			
Extra Large Adult	16+ years	60 – 120	6		 Code: 7194000
	16+ years	> 120			

2

Remove QuadraLite from the packaging and attach to the breathing system. Position the face mask with the base of the mask sitting in the lower lip fold so that it covers the mouth and nose. The anatomical shape will conform to a wide range of face sizes and shapes.

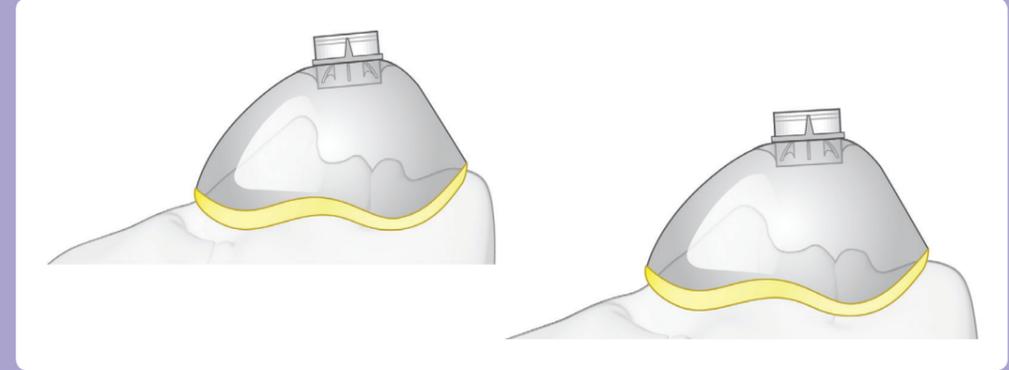

3

Hold the mask using your thumb and index finger, leaving the remaining fingers to perform a chin lift maneuver if required.



4

It is not necessary to press down hard. The materials used and the truly anatomical shape mean that a seal is achieved with minimal pressure.



5

QuadraLite™ is a single use product and should be discarded in accordance with hospital protocols following use. QuadraLite™ is part of the Intersurgical Eco range of products as it does not contain PVC and therefore it also does not contain phthalates.



For more information please visit:  
[www.intersurgical.com/info/quadralite](http://www.intersurgical.com/info/quadralite)



Quality, innovation and choice



Interact with us  
[www.intersurgical.com](http://www.intersurgical.com)